EN The Ellis® MC34 Connector is to be used exclusively in combination with the BADAL X Humeral Implant; the Ellis® LS Loadset or the Ellis® A2/A3/A4 mechanical elbow with end terminal. The device is intended to function as a fixation of the loadset / arm prosthesis to the implant. This package contains:

- Ellis® MC34 Connector [1]
- User instruction

Important - Prior to any use, the Ellis® MC34 connector is intended to be fixated to the Ellis® LS Loadset or to the Ellis® A2/A3/A4 mechanical elbow with end terminal. The device is to be fitted to the patient and to be used under guidance / responsibility of a certified prosthetist. For use of this device with the Ellis® LS Loadset, a loading protocol applies; for more details see the IFU of the Ellis® LS Loadset.

Calibration protocol - The Ellis® MC34 Connector should be attached to the Ellis® A2/A3/A4 mechanical elbow in such a position, that the prosthetic arm obtains a neutral position. The prosthetist can adjust the position of the connector by use of an allen key (detach and rotate the connector relatively to connector base into the correct position (by small steps $\pm 6^\circ$).

Instruction for use of this device

1. Donning and doffing of the prosthesis (or loadset)

When fixating to the implant; make sure the connector is 'opened' [1a].:

[1a] Prepare: OPEN the connector

Lift turn ring of the connector [2] and rotate it (in correct direction) to OPEN the connector. **The turn ring is now 2 mm lifted and the connector is in 'opened' state.** (See image at [1a] below).


[1b] + [1c] POSITION correctly, then FIXATE

Position BADAL X Humeral implant with the domed cam [3] above the connector slot; slide the connector all the way up and rotate in the correct direction to FIXATE. The connector automatically locks when you let go of the ring.

[1c] Detach from BADAL X implant

Lift the turn ring [2] and rotate it to obtain 'OPEN' state. Then slide down the arm prosthesis and detach from the implant. Be sure to keep hold of the device while disconnecting to avoid it falls down.


[1a] Prepare: OPEN the connector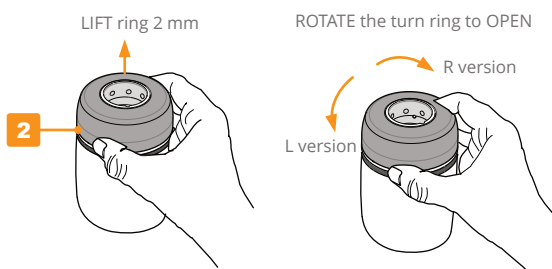

Rotate turn ring to the right to open a right version (R)
 Rotate turn ring to the left to open a left version (L)
 OPEN = turn ring remains in uplifted position (2mm);
 >> connector is in 'OPEN' state


[1b] POSITION correctly on implant

Attach connector to the implant by matching domed cam to connector-slot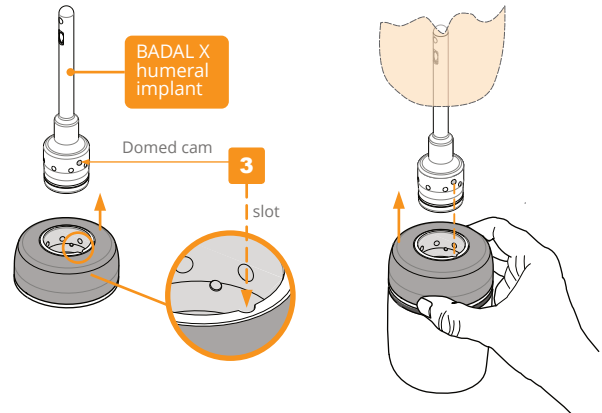

Position the 'opened' connector by sliding it at the correct position all the way upward over implant ... hold the prosthesis ... >> (see below)


[1c] ROTATE to FIXATE to implant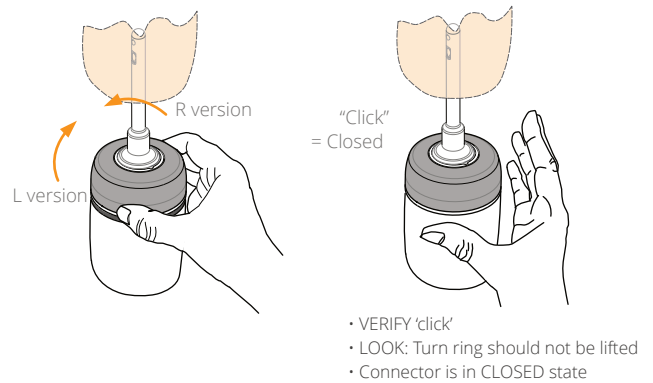

>>> ROTATE in other direction to FIXATE to the implant :

- Rotate to the right to fixate a LEFT version (L)
- Rotate to the left to fixate a RIGHT version (R)

[1d] DETACH from implant

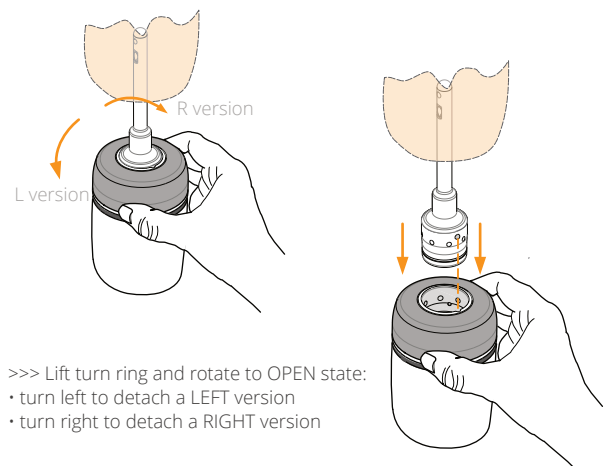- >>> Lift turn ring and rotate to OPEN state:
- turn left to detach a LEFT version
 - turn right to detach a RIGHT version

Keep a hold of the 'opened' device while pulling it down >
 Then detach the arm prosthesis from the implant ...

Further specifications are mentioned on backside of this IFU..

SPECIFICATIONS

Classification:	Class I medical Device - low risk (MDR:2017/745)
Materials:	3D-printed Nylon, stainless steel, aluminum
Weight:	0.3 kg

Warranty

A 2 year warranty on materials and mechanism and construction parts is applicable; provided this device is used as intended. The intended use includes following load limitations:

Loading limitations (*)

- Maximum hand weight in mode 90° flexed elbow : 8 kg
- Maximum weight on extended elbow (*neutral shoulder position*): 15 kg

(*) *These loading limitations only apply to the arm prosthesis; the arm prosthesis has been tested for above weight. Please inquire with your prosthetist or surgeon for information on applicable loading limitations in your specific situation.*

Intended use / target group / exclusions / cleaning

This device is to be used by humeral amputees with a BADAL X humeral implant and is intended to function as a connector for prostheses of the type Ellis LS Loadset or Ellis® mechanical elbow prosthesis with end terminal. The intended use of the device excludes high impact and loading activities. The construction is watertight; when using the device in sea or swimming pool, make sure to thoroughly clean with a humid cloth (not wet) after each use. Avoid to splash water at the INSIDE of the connector.

Compatibility

- 1284/1284E - Ellis® A2 mechanical elbow (2 fixations)
- 1453/1453E - Ellis® A3 mechanical elbow (3 fixations)
- 1380/1380E - Ellis® A4 mechanical elbow (4 fixations)
- 1354/1354E - Ellis® CBESP/CBESP-E connector base (Espire)
- 1389/1354E - Ellis® CB12K/CB12K-E connector base (Ottobock type 12K)
- 1558/1558E - Ellis® MOTUT / MOTUT-E connector base (Motion/Utah)
- 1390 - Ellis® CBUNF connector base for UNF 1/2x20 thread

Contraindications

- inability to commit to end-user instructions of this device

Residual risks

- unintended overloading

Training

A loading and training protocol is applicable prior to use of this device, by guidance of a certified prosthetist and surgeon.

Warnings

This product is explicitly not intended to be used in high-impact conditions (e.g. extreme sports such as skiing, mountain biking, motocross/biking or water skiing etc.) as this may lead to overloading of implant or breakage of the exo-prosthesis and cause further injury or other damage. Any damage resulting from such use is at the user's risk. Do not overload the device and eliminate contact of this connector with sand as much as possible. Do not combine the device with other components than those mentioned on this IFU for risk of malfunctioning or loss of warranty. If off-label use is intended, this must be reported to the manufacturer in order to evaluate possible risks. Without the manufacturer's written permission for off-label use, warranty is not guaranteed. DHM-medical is entitled to limit the warranty in some off-label use cases. Do not reuse and combine separate components without written approval of the manufacturer in order to avoid loss of warranty.

Maintenance

Clean the connector on a regular basis (daily) by use of water and soap on a humid cloth. In case of sandy environment, make sure to remove any sand in the connector by shaking the connector upside down. Do NOT use tap water inside the connector. After a period of time, internal fixation balls may need to be exchanged. The functionality and adequacy of this device is to be monitored by the prosthetist on a regular basis, at minimum once a year.

Durability and expected lifetime

This connector is protected from total dust ingress and waterproof (IP67). (THIS ONLY APPLIES TO COMPLETE ASSEMBLED DEVICE WHICH IS CONNECTED TO BADAL X IMPLANT WITH ELLIS A2/A3/A4 ELBOW AND IPX7 END TERMINAL) The life span for normal use is estimated to last about 4 years or 10.000 cycli or whichever applies first. The materials are resistant

to regular cleaning detergents (*alcohol, water, soap*). However, frequent exposure to salt water or chlorine may lead to material wear / deterioration of the device, which is excluded from the warranty.

Optional parts - spare parts

No spare parts apply for this device.

Malfunction

This device is designed to function but nonetheless a malfunction may occur in which case following advice is provided:

- If the connector attachment to the implant is malfunctioning; clean the inner connector surface and retry. If this does not help, contact the prosthetist or surgeon.
- If the connector detachment from the implant is malfunctioning; contact the prosthetist or surgeon.

Reuse - recycling - waste

This device is not intended to be reused after its life-span. Its disassembled components can be disposed of by the user through the regular waste disposal channels. The manufacturer has implemented a recycling protocol; devices sent back to the prosthetist; will be collected on a periodical basis and shipped back to the manufacturer in order to be disassembled and recycled.

Calibration-protocol

In case the connector-implant slot-cam fit results into an incorrect position of the prosthesis (*not neutral*), a connector-calibration protocol applies. This calibration protocol is to be conducted by a certified prosthetist. No risks apply for the end user or prosthetist, when conducting this calibration.

Temperature / humidity

Normal usage temperature range applies: 0°C - 40°C.

No restrictions for humidity apply for the device.

Do not expose this device to high temperatures or to extreme cold and frost.

Use in a therapeutic or research environment

It is advised to detach this device when a user is subject to clinical examination (i.e. MRI). The metal parts present in this device may cause unintended alarm at security gates.

Use of consumables

No use of consumables applies for this device.

Vigilance

DHM-medical is convinced to have done everything possible to safely and reliably market this device. To this end, all guidelines have been followed, discussions with experts have been held and ISO 13485 QMS including GMP has been conducted. If despite all this you believe this device does not meet the qualities / standards that are to be expected, you can report this to the appropriate authorities.

This IFU has been compiled with care. If you have any comments please contact us at info@ellis-prosthetics.com.

ELLIS Prosthetics is a registered trademark of DHM-medical B.V. Netherlands.

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ELLIS Prosthetics jest zastrzeżonym znakiem towarowym firmy DHM-medical B.V. w Holandii.