
EN

This rotational wrist + sleeved active mHand Adapt is exclusively to be used by humeral amputees in combination with a BADAL X Humeral Implant; Ellis® MC34 Connector and Ellis A2-3-4 mechanical elbow. Intended use: Therapeutical aid for lack of wrist / hand functions. Package contains:

- (internal) rotational friction wrist [1];
- (internal) mHand Adapt [2] offering direct grasp / indirect grasp; with adjustment of the grip width by use of a release button [3]
- (assembled) functional cover sleeve - mHand Skin [4]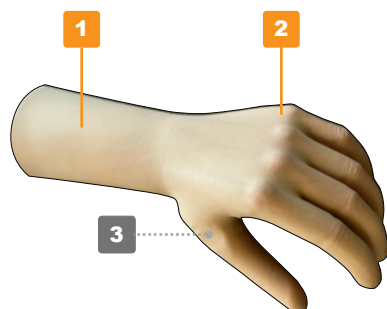

**Important** Prior to any use of this device in combination with humeral BADAL X implant, make sure that the BADAL X load protocol has been completed. It is essential to allow the humeral bone sufficient time to heal after the surgery - for this purpose a special loading protocol applies, which is to be guided by a certified prosthetist and surgeon.

Ellis® B2 Wrist + sleeved active mHand is to be ordered under responsibility of a certified prosthetist. The device has a patient specific design; matching the dimensions and functions of the users' sound contra-lateral hand, and is prescribed by the prosthetist.

## Instruction for use of this device

### 1. Wrist rotation - fixation of end terminal

All Ellis® end terminals can be freely rotated by use of an adjustable friction-rotated movement from +125° (pronation) to -125° (supination). In order to detach the end terminal, exo-rotate the end terminal to the maximum supination position (*palm upwards*); then detach the end terminal (*pull*) from the forearm [2].

To re-attach, make sure to place the end terminal in correct position and push it back onto forearm and then rotate back in a neutral position [1].

[1]

## SPECIFICATIONS

Classification: Class I medical Device - low risk (MDR:2017/745)  
 Materials: PVC (sleeve), PA (wrist; mHand), metal (screws)  
 Weight: (average) 455 g

## Warranty & Expected Lifetime

A 2 year warranty on materials for the wrist / internal construction parts applies, provided the device is used as intended. Expected lifetime of wrist fixation is 5 years or 10 000 cycles of service, whichever is first. Expected lifetime of end terminal mHand Adapt is 5 years or 100 000 cycles of service, whichever is first. Usage beyond these limits might lead to failure and dropping of the carried object. The sleeve is a wear part; subject to normal wear and tear for which no specific warranty applies.

## Intended use / target group / exclusions / cleaning

This device is intended for use by humeral amputees, as a wrist with active hand. Intended use excludes high impact / loading activities. Avoid getting water and/or sand on INSIDE of the wrist connection mechanism.

## Contraindications

- inability to commit to end-user instructions of the device

## Residual risks

- unintended overloading

## Training

Intended use / training protocol for correct use of wrist and hand functions is to be conducted under guidance of a certified prosthetist.

## Loading limitations (\*)

- Maximum wrist weight in mode 90° flexed elbow : 8 kg
- Maximum weight on mHand Adapt: 10 kg (*lift with straight arm*)

(\*) For end users: Please inquire with your prosthetist or surgeon for information on loading limitations in your specific situation.

## Special instructions use of mHand Adapt

We hereby refer to the IFU of mHand Adapt (LM Moveable, Netherlands); included in the packaging of this device. A summary is given below:

- Prevent dirt, sand, dust and moisture from entering hand mechanism to avoid function loss. In case the hand mechanics gets wet, make sure it's completely dry when using it again.
- To adjust thumb friction, use a 3 mm hex key and adjust the screw on top of thumb. (*Located between pointing finger and middle finger > easiest to reach when hand is in closed position*)
- Always use mHand combined with sleeve to enable effectively grasp objects. Without the sleeve, malfunction / damage can be expected.

## Compatibility

- 1284 - Ellis® A2 mechanical elbow (2 fixations)
- 1453 - Ellis® A3 mechanical elbow (3 fixations)
- 1390 - Ellis® A4 mechanical elbow (4 fixations)

## Special instructions for maintenance / cleaning of Sleeve

We hereby refer to the IFU of mHand Skin [8S18] which is separately included in the packaging of this device. A summary is given below:

### Light soiling (use lint-free cloth; water and soap)

- 1 Clean the product with warm water, soap and a cloth
- 2 Rinse off soap residue with water
- 3 Dry the product with a lint-free cloth and allow it to air dry fully

### Heavy soiling (use 640F12 cleaner, pump sprayer 640F13, lintfree cloth)

- 0 Clean promptly to avoid permanent settling of soil in sleeve material
- 1 Spray cleaner on soiled area and allow brief effect taking (*wait a bit*)
- 2 Wipe off the soiling be gently pressing with a clean undyed lintfree cloth
- 3 Dry the product with a lint-free cloth and allow it to air dry fully

## Disposal sleeve

Do not dispose this sleeve with unsorted household waste.

## Warning sleeve

Keep sleeve material away from open flames, embers and other sources of heat. Do not expose the sleeve material to oils, salves, lotions or other products that contain oils or acids.

## General Warnings

This device is explicitly not intended to be used in high-impact conditions (e.g. extreme sports such as skiing, mountain biking, motocross/biking or water skiing etc.) as this may lead to overloading of implant or breakage of the exo-prosthesis and cause further injury or other damage. Do not overload the mHand by 100 000 cycles; respect also the load limitations and instructions of the prosthetist to minimize risks.

Limit contact of the wrist rotation mechanism with sand as much as possible to avoid the risk for mechanical malfunction of the fixation. Do not soak the disconnected device under water to avoid the risk of mechanical wear. Do not reuse and combine separate components without written approval of the manufacturer in order to avoid loss of warranty.

## Maintenance

Clean the device on a regular basis (*daily*) by use of water and soap on a humid cloth. In case of sandy environment, make sure to remove any sand in the elbow and wrist mechanism by detaching the device and shaking it in upside down position. Do not use running water inside the device components (*elbow / wrist / hand*). Functionality and adequacy of this device is to be monitored by the prosthetist on a regular basis, at minimum once a year.

## Durability

This device is dust-free and waterproof (IP65). **SWIMMING WITH THIS DEVICE IS EXCLUDED FROM WARRANTY.** The materials used are resistant to regular cleaning products (*water, soap*). However, if the device is frequently exposed to salt water or chlorine environments, this may lead to material wear / deterioration, which is excluded from the warranty.

## Optional parts - spare parts

Exchange parts are available - ask the prosthetist or inquire with the manufacturer for exchange options.

## Malfunction

This device is designed to function correctly, however a malfunction may occur in which case advice is provided below:

- *wrist attachment is malfunctioning*  
If the wrist attachment is malfunctioning; detach it; clean the inner connector surface by use of a humid (not wet) cloth and retry. If this does not help, contact the prosthetist.
- *mHand does not close when trying to grasp an object*  
Hand closure is directly dependent on backward extension of the wrist. In order to extend the wrist, the direction of the load on the hand must be perpendicular to the stump. Be sure to either press against thumb or against palm of the hand, and not against the fingers!
- *Thumb rotates when trying to grasp an object*  
Increase the thumb friction as described in •• above (*Special instructions use of mHand Adapt*).

## Reuse - recycling - waste

This device is not intended to be reused after the life-span. Its disassembled components may be disposed of by the user through the regular waste disposal channels. In addition to this, the manufacturer has implemented a recycling protocol; devices sent back to the prosthetist will be collected on a periodical basis and shipped back to the manufacturer in order to be disassembled and recycled.

## Calibration

Calibration may apply to the friction force of the wrist rotation which is adjustable by use of an Allen Key. Furthermore the thumb rotation is adjustable, see above at mHand instructions for use.

## Using temperature / humidity

Normal usage temperature range is 0°C - 40°C.  
No restrictions for humidity apply for this device.  
Do not expose this device to high temperatures or to extreme cold and frost.

## Use in a therapeutic or research environment

It is advised to detach the prosthesis when a user is subject to clinical examination (i.e. MRI). The metal parts present in the device may cause unintended alarm at security gates.

## Use in therapeutic or research environments

It is advisable to detach the device when a user is undergoing a clinical examination (e.g. MRI). Metal parts inside the device may cause an involuntary alarm at security gates.

## Use of consumables

- Sleeve used is subject to normal wear and tear. Replace it when necessary and keep in mind that a leak sleeve may cause the mHand to malfunction

## Vigilance

DHM-medical is convinced to have done everything possible to safely and reliably market this device. To this end, all guidelines have been followed, discussions with experts have been held and ISO 13485 QMS including GMP has been conducted. If despite all this you believe this device does not meet the qualities / standards that are to be expected, you can report this to the appropriate authorities.

This IFU has been compiled with care. If you have any comments please contact us at [info@ellis-prosthetics.com](mailto:info@ellis-prosthetics.com).

[Ellis-prosthetics® is a registered trademark of DHM-medical B.V. Netherlands.](#)

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[Ellis-prosthetics® jest zastrzeżonym znakiem towarowym firmy DHM-medical B.V. w Holandii.](#)