
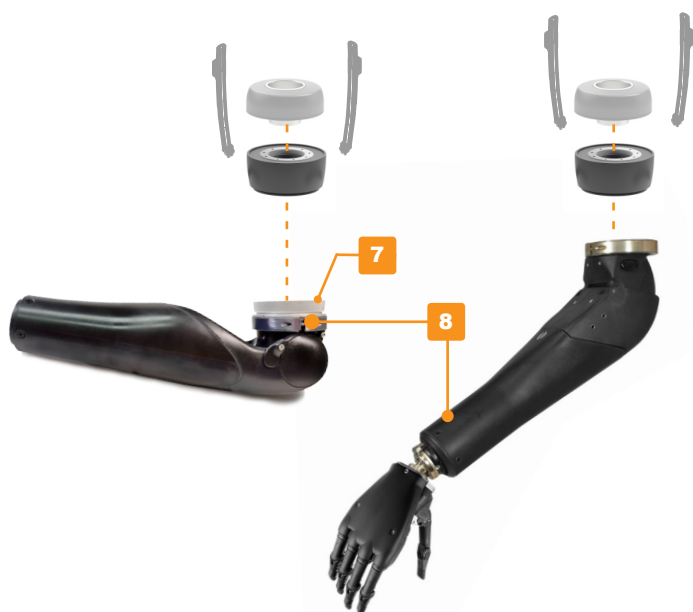
**EN** Please read user instructions before use. This device is intended for exclusive use by humeral amputees with a BADAL X humeral implant in combination with a clamp [7] for fixation of the (myo) elbow of type Motion Arm / Utah Arm [8] and to be combined with 1644 - ELOC electrode positioning holder [6] and 1097 - Ellis® MC34 Connector [5] - (not included.) Packaging contains :

- ELLIS MOTUT-E Connector-base [1]
- Fixation ring [2] + 9x screw [3]
- 1 user instruction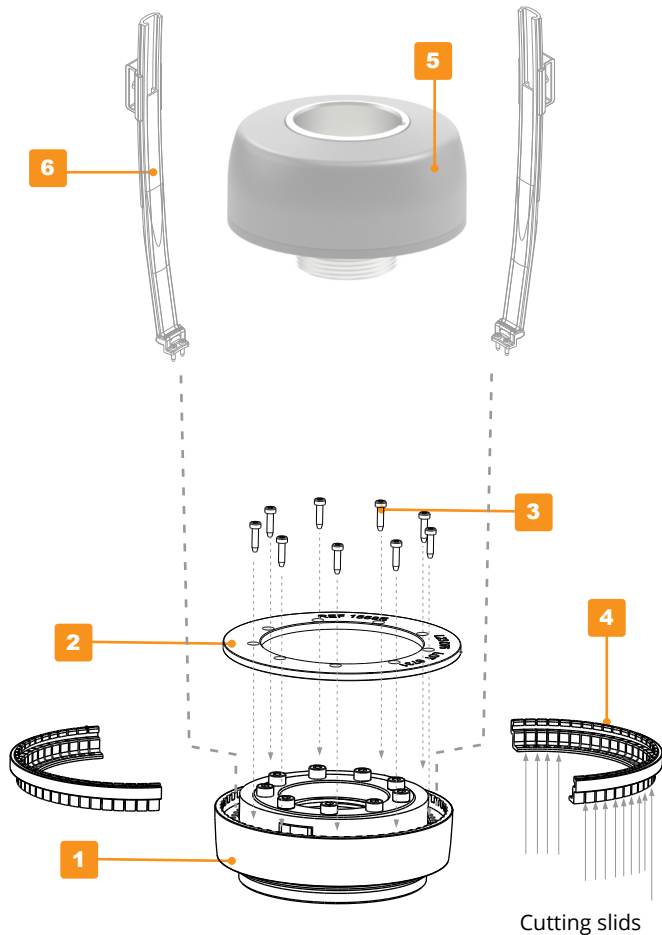

Left: Motion Arm (Fillhauer) with Clamp fixation (purchase separately)  
Right: Utah Arm (purchase separately)

### Instruction for ELLIS MOTUT-E

1. Mark location of electrodes on patients' stump.
2. Place each ELOC positioner(s) [6] onto the MOTUT-E Connector-Base [1] at estimated position(s) (DO NOT YET SCREW)- slide on each electrode clip and mount each electrode at estimated height; loosely fixate the electrode clip (screw); let all cables 'hanging' (DO NOT YET GUIDE / COVER CABLES!)
3. Mount MC34 Connector [5] to MOTUT-E Connector-base [1] (Do NOT YET place cover ring [2] and screws [3]).
4. Verify on patient if the configuration can obtain correct signals from the stump - adjust positioner holders if necessary (*lift - rotate / slide to adjusted location*). Adjust location of electrode clips if necessary (*descrew - slide - screw*). When verified; shorten positioner holder just above the electrode clip and round the tip. Mount Velcro Strap through Strap clips - verify for correct functioning of the configuration (donning/doffing). (Details: DS1644\_IFU\_ELOC)
5. Detach MC34 Connector in order to fixate position holders at the bottom (*use the screws*). Use fill-rings [4] to close gaps between the holders: cut to size (*indicated slits*) until all gaps are filled.
6. Guide electrode cables inside MOTUT-E by use of nearest hole in the connector-base. Cut cable cover lid *below electrodes* and mount.
7. Place cover lid [2] onto Connector-base [1] and fixate screws [3].
8. Verify for sufficient space to mount the control unit of the myo-device inside the connector-base (*if the control unit you purchased is too big find another location in the prosthesis to mount it*).
9. Re-attach Ellis MC34 Connector onto MOTUT-E connector-base. Check for neutral position of arm prosthesis (\*).
10. Fixate exo-prosthetic arm [8] by use of clamp [7] fixation (*order at third party*)

(\* If necessary for correct neutral position of exo-prosthetic arm, adjust the basic orientation of the Ellis® MC34 connector. For details see IFU of Ellis® MC34 connector).

**IMPORTANT NOTE** - never overload the implant - loads applied on the system should not exceed limits for MC34 connector.

### End user instructions

**Maintenance** - Visit the prosthetist on an annual basis for functional checks of your prosthesis. This device should be protected during use from solvents, dust and dirt. Use of water/ soap to clean it.

**Consumables - spare parts** - No consumables apply. Spare items can be purchased at the manufacturer; refer to indicated article numbers.

**Single use** - This device is a Single use device - do not reuse it.

**Sterilization** - NO sterilization applies prior to use.

**Materials** - This device is produced from PA12, durable plastic with a heat resistance temperature of 180°C; biocompatibly safe and FDA food proof. NO Latex is used in this device.

**Malfunctioning or damage** - If the Adapter is malfunctioning, stop use of the device and contact your prosthetist.

**Compatibility** - This device is compatible with Ellis® MC34 Connector. Do not use in combination with other connectors - avoid unforeseen hazards.

**MRI** - Take of your prosthesis when subject to MRI or other therapeutical environments that may be influence by metal.

**Residual risks** - No residual risks apply.

**Warnings and precautions** - It is important to practice the donning/doffing routine. Inquire with your prosthetist for training tips.

**Clinical benefit** - this device supports for detachable donning/doffing of an exo-prosthetic elbow from a humeral implant of the type BADAL X.

**Contraindications** - at too long stumps the remaining build height is insufficient (*min. 150 mm from bottom implant to elbow*)

**Storage / Usage temperature** - no special storage temperature applies - Usage temperature is between 0°- 40°C.

**Disposal** - After stop use of this device, it should be disassembled and disposed as separate waste.

**Vigilance** - DHM-medical has done all possible to produce a safe product. If despite you are the opinion this product is unsafe you should contact your competent Authority.

**Downloads** - [www.ellis-prosthetics.com/downloadpage](http://www.ellis-prosthetics.com/downloadpage)

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ELLIS Prosthetics is a registered trade mark of DHM-medical B.V., Netherlands
DHM-medical B.V.